



	Instructions for use	Doc. No.: IU/25
	For	Rev. No.: 01
	Paraffin Gauze Dressing	Rev. Date: 13SEP2024

Descriptions:

Paraffin Gauze Dressing is a yellow soft paraffin dressing that is not medicated, making it ideal for use with topical antibiotics or antiseptics. It is soothing and low-adherent, and allows the wound to drain freely into an absorbent secondary dressing. It is available in a wide range of sizes, the largest of which is suitable for use in burns units.

1. Indications:

Paraffin Gauze Dressing is used as a primary wound contact layer in the treatment for

- Minor burns and ulcers.
- Skin grafts (both donor and receptor sites).
- Variety of traumatic injuries.

2. Contraindications:

Patient sensitive to soft paraffin should not use the product. Paraffin Gauze Dressing such has no contraindication. However, it is rare occurrence if the dressing used for heavy exuding wound the semi-occlusive nature cause maceration by preventing movement of exudates away from the surface of the wound.

3. Precautions:

Seek medical advice immediately for all serious wounds and burns or if redness or discomfort occurs. Change dressing daily or more often according to the condition of the wound. Device should not be re-used. Cross-contamination can occur and the therapy may be ineffective. Opening the dressing pouch compromises the sterile barrier, therefore any unused dressing should not be retained for later application. Avoid contact with eyes as it may cause irritation. Dressing contains flammable substance: Paraffin. Avoid direct contact with naked flames or sources of ignition.

4. Presentation

Each dressing (Including the roll and box) is wrapped with vegetable parchment paper in an individual foil pouch, Box and sterilized by gamma radiation.

5. Packing and labeling:

The Dressings are supplied in sterile condition; packed in individual aluminum foil pouches, polypropylene boxes and kept inside suitable size of outer box. Sterilization is carried out using the gamma irradiation process indicated by **STERILE R** on the label.

6. Storage condition:

Store flat in a dry place at a temperature not exceeding 25°C.
Protect devices from exposure to direct sunlight, heat and rain.



	Instructions for use For Paraffin Gauze Dressing	Doc. No.: IU/25
		Rev. No.: 01
		Rev. Date: 13SEP2024

7. Symbols Used in IFUs/Labels and Packaging Materials

Symbol	Definition	Symbol	Definition	Symbol	Definition
	Single use (Do not re-use)		Batch Number		Keep away from heat /sunlight and radioactive sources
	Date of Manufacture YYYY-MM		Manufactured by		Use by Date (Date of Expiry) YYYY-MM
	Caution: check for specific warnings or precautions		Do not re-sterilize		Do not use if package is opened or damaged
	Sterilized by Gamma Sterilization process		Consult instructions for use		Avoid moisture or water contact
	Recycle		Code Number / Part No.		To be sold only against prescription

8. Further information:

For further information concerning use of these devices, please check with Adler Customer Service at the address given herein or e-mail to adler-customer.care@adler-healthcare.com.

Manufactured by:



Adler Healthcare Pvt. Ltd.

Plot No. A-1, MIDC Sadavali, Tal. Sangameshwar, Dist. Ratnagiri, Maharashtra State, INDIA,
 PIN 415804. Ph: +91 (0)8055136000, 8055146000